Respirator Fitting Instruction

STEP 1

Hold the respirator in hand with the nosepiece at your fingertips, allowing the headbands to hang freely below your hand.

STEP 2

Press the respirator firmly against your face with the nosepiece on the bridge of your nose.

STEP 3

Stretch and position the top band high on the back of the head. Stretch the bottom band over the head and position below your ears.

STEP 4

Using both hands, mold nosepiece to the shape of your nose.

STEP 5

To test fit: a) Cup both hands over the respirator being careful not to disturb position, and b) inhale vigorously. If air leaks around the edges, reposition the straps or adjust strap tension for better fit.

If there are still noticeable leaks then this mask may not be suitable for you and we advise you to seek another style from our range.

Service Time Limitation

1. Do not use the same respirator for more than 8 hours, continuous or intermittent, in dirty workplaces that could result in high filter loading.
2. Leave contaminated area and remove respirator if breathing becomes difficult or proper fit cannot be obtained or the respirator is damaged, soiled, or distorted.
- Do not use the same respirator again once removed.

CE 2834 FFP2 NR
EN149:2001+A1:2009
Quality Management System in line with
ISO 9001:2015 Standard

Warnings Read and understand before use

1. A properly selected respirator is essential to protect your health. Before using this product consult a safety professional to determine its suitability for your intended application.
2. As with any respiratory device, the wearer MUST be adequately trained prior to use. Before use, the wearer should always check the mask is in good condition - not dirt, no damage to headstrap attachment etc.
3. This product does not protect the wearer against gases, vapors or mist whose liquids produce harmful vapors or gases.
4. Do not use in sandblasting operations.
5. This product does not supply oxygen. Use this respirator only in well-ventilated areas containing sufficient oxygen (at least 19.5%) to support life.
6. Do not use when contaminant and the concentration are unknown or immediately dangerous to life and health.
7. Leave work area immediately if:
a) breathing becomes difficult,
b) dizziness or other distress,
c) the respirator becomes damaged or d) an emergency occurs.
8. If user has any illness or discomfort, consult a licensed medical doctor to determine suitability of use.
9. Facial hair and certain facial characteristics will prohibit effective use of this product.
10. Replace a respirator when the breathing resistance becomes excessive due to dust loading or after a maximum of 8-hours continuous usage, "single shift use".
11. Failure to observe all warnings and instructions on the use of this respirator may result in serious illness or permanent injury.
12. The equipment should not act as an ignition source but it should not be used in explosive atmospheres as the concentration of contaminants within normal flammability range would exceed the limits in the selection guide.
13. Store respirator in a sealed container away from contaminated areas when not in use.
14. Store respirator in the original packing during transport.
15. Please make sure the mask fits properly before entering working environment.

The filtering respirator is designed for applications involving fine non-toxic dust and fibers. This respirator meets the requirements of EN149:2001+A1:2009 (FFP2 NR). These devices are designed to protect against both solid and liquid aerosols. The respirator can be used in concentrations of contaminants

up to 20 times the Occupational Exposure Limits (OEL).
Manufactured by: Champion Industry Corp. Ltd
Address: No.315, Zhongshan South Road, Nanjing City, Jiangsu, China
These respirator products meet the requirements of Regulation (EU/UKCA) 2016/425 and are CE marked accordingly.
They have been tested according to EN 149:2001+A1:2009. Module B EU Type-Examination and Module D Certification have been issued by: CCQS Certification Services Limited Block 1 Blanchardstown Corporate Park, Ballycoolin Road, Blanchardstown, Dublin15, D15 AKK1, Ireland (NB 2834) UKCA approved body: CCQS UK Ltd. (AB 1105), 5 Harbour Exchange Square London, E14 9GE, United Kingdom. EU/UKCA.

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